

# MEDI DERMA-S

## Total Barrier Film - Non-Sting

### Assessment & Record Chart



Patient's Name:	<b>Type of Moisture Associated Skin Damage (MASD):</b> Incontinence <input type="checkbox"/> Intertrigo (Skin Folds) <input type="checkbox"/> Periwound <input type="checkbox"/> Peristomal <input type="checkbox"/>
Start Date:	
Assessed by:	
Comments:	
<b>REGULARLY ASSESS THAT THE APPROPRIATE PRODUCT IS BEING APPLIED</b>	

#### Products Used:

- Aerosol - 50ml
- Pump Spray - 30ml
- Wipe
- Sterile** Applicator - 1 ml
- Sterile** Applicator - 3ml

#### Frequency of Application:

- At each dressing change
- Every 24h
- Every 48h
- Every 72h
- Other (Please detail):

#### Instructions for Use:



**1**

**Skin Preparation:**

- Cleanse the affected area and pat dry.



**2**

**Application:**

**Aerosol/Pump Spray**

- Hold away from the skin surface  
Aerosol 10-15cms.  
Pump Spray 5cms.
- Spray using a slight sweeping movement.



**3**

**Application:**

**Applicator/Wipe**

- Apply using a slight sweeping movement.
- Apply to surrounding skin only, when use for management of periwound or peri-stomal maceration/excoriation.



**4**

**Apply Dressing/Device:**

- Once fully dry\*, dress the affected area, if applicable.

**\*Drying Time**  
 Aerosol or pump spray: 30 seconds  
 Applicator or wipe: 5-10 seconds  
 Hold apart skin folds to ensure complete drying.

#### Area of the Body Being Treated:

Application Site/s: \_\_\_\_\_  
 (Mark with an 'X' where appropriate)



